

# ORTHOCOR ACTIVE SYSTEM SIZING CHART

Measure the circumference around the center of the treatment area or use patient's weight or shoe size to determine which size is appropriate for the patient.

## SHOULDER\*

Measure the circumference around the patient's chest and shoulder starting under the arm, around the back, and over the shoulder.

**One Size Fits All**

|                 |                 |
|-----------------|-----------------|
| Min. Length 20" | Max. Length 36" |
|-----------------|-----------------|

\*XS, S, L, XL Extension Straps Available

## NECK

The neck device does not require measurement as the straps only come in one length.

**One Size Fits All**

## HAND/WRIST

Measure the circumference around the patient's wrist.

|        | Small     | Large     |
|--------|-----------|-----------|
| Length | Below 11" | Above 11" |

## ELBOW

Measure the circumference around the middle of the patient's elbow with a slight bend.

|        | Small        | Large        |
|--------|--------------|--------------|
| Length | 9" - 12"     | 12" - 16"    |
| Weight | Below 200lbs | Above 200lbs |

## BACK\*

Measure the distance around the patient's low back.

|        | Small        | Large        |
|--------|--------------|--------------|
| Length | 27" - 38"    | 38" - 51"    |
| Weight | Below 200lbs | Above 200lbs |

\*Universal 16.25" Extension Straps Available

## KNEE\*

Measure the circumference around the middle of the patient's knee.

|        | Small        | Large        |
|--------|--------------|--------------|
| Length | 13" - 16.5"  | 16.5" - 20"  |
| Weight | Below 200lbs | Above 200lbs |

\*Universal 16.25" Extension Straps Available

## FOOT

Measure the circumference around the top of the patient's ankle.

|                 | Small     | Medium    | Large |
|-----------------|-----------|-----------|-------|
| Length          | 10" - 13" | 13" - 16" | 16"+  |
| Mens' Shoe Size | 5 - 9     | 10 - 12   | 13+   |

## ANKLE

Measure the circumference around the top of the patient's ankle.

|                 | Small     | Large |
|-----------------|-----------|-------|
| Length          | 10" - 15" | 15"+  |
| Mens' Shoe Size | 5 - 11    | 12+   |

## UNIVERSAL/HIP\*

This device is used to treat areas of the body where there is not a specific device, such as the hip. Measure the circumference of the injured area.

**One Size Fits All**

|             |                 |                 |
|-------------|-----------------|-----------------|
| Short Strap | Min. Length 8"  | Max. Length 10" |
| Long Strap  | Min. Length 20" | Max. Length 36" |

\*XS, S, L, XL Extension Straps Available

### Extension Straps Lengths on Request

|           |           |           |           |
|-----------|-----------|-----------|-----------|
| <b>XS</b> | 6" - 7"   | <b>L</b>  | 21" - 37" |
| <b>S</b>  | 7" - 8.5" | <b>XL</b> | 33" - 62" |

### ADDRESS & IMPROVE

- Post-operative Pain
- Acute & Chronic Pain
- Arthritis
- Sprains & Strains
- Inflammation
- Edema

### RISK-FREE SOLUTION

- Non-narcotic pain management
- Wearable pain relief
- Accelerates natural healing
- Daily, in-home treatments



#### ORTHOCUFF

Fully adjustable design for ultimate fit & mobility


#### ORTHOPODS

Patented, Disposable Pods Provide Heat & Initiate Therapy

#### ORTHOSOURCE

Easy to Use 2-Hour Daily Treatment

Neck Active System

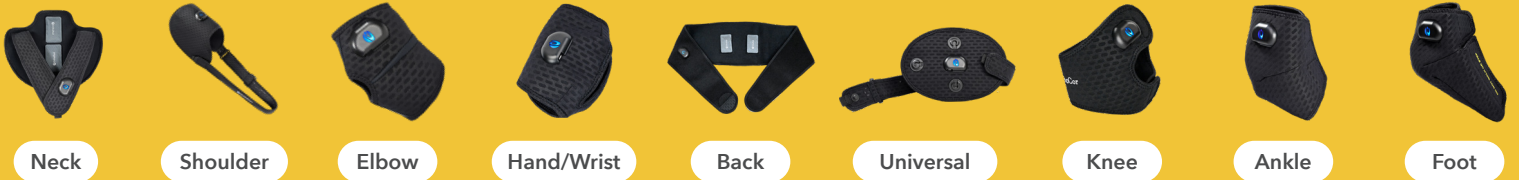

## STATE OF THE ART TECHNOLOGY: PULSED ELECTROMAGNETIC FIELD (PEMF) THERAPY

The OrthoCor Active System utilizes Pulsed Electromagnetic Field (PEMF) therapy to mitigate pain and encourage natural healing. The OrthoCor Active System accomplishes these results at the cellular and molecular levels through a clinically proven mechanism of action. PEMF therapy increases nitric oxide (NO) in the treatment area, which results in increased blood and lymphatic flows, as well as the inhibition of inflammatory agents including COX-2 and interleukin 1 $\beta$ .

